



## OVAL SENSOR DRESSING

Use with OVAL PressureAlert® Sensor



Pull back top flap of the dressing and insert **OVAL SENSOR** (battery side up).



Reseal the top flap and turn dressing over to remove the bottom adhesive liner.



Loosely apply the middle of the dressing over the bone; firmly smooth flaps to skin one at a time.

**SENSOR IS SINGLE PATIENT USE ONLY**



**Digital Health Solutions**

(855)727-0347

WWW.DIGITALHEALTHS.COM