



## SACRAL SENSOR DRESSING


Use with ROUND PressureAlert® Sensor



1 Pull back top flap of the dressing and insert **ROUND SENSOR** (battery side up).



2 Reseal the top flap and turn dressing over to remove the bottom adhesive liner.

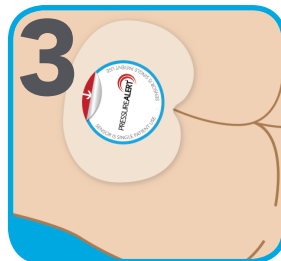

3 Apply the middle of the dressing onto skin over the sacral bone, then smooth sides down.

**SENSOR IS SINGLE PATIENT USE ONLY**



 **Digital Health Solutions**

(855)727-0347

WWW.DIGITALHEALTHS.COM